



PHAGO

PHAGO'SPRAY DASR 70

No-rinse hydroalcoholic disinfectant for hands and surfaces.



- Broad antimicrobial spectrum from 30 s
- Food contact, no rinsing
- No trace or residue when drying

USE

PHAGO'SPRAY DASR 70 is a hydroalcoholic disinfectant for disinfecting hands, surfaces, equipment and clean furniture. It leaves no traces or residues on surfaces. It can be applied to a single-use wipe for rapid disinfection of small items of furniture, trolleys, external surfaces of non-invasive medical devices (operating table, operating lights, monitors, cables and connectors, stethoscopes, reflex hammers, blood pressure gauges) and small surfaces close to the patient.

PHAGO'SPRAY DASR 70 can be used, without rinsing, on surfaces that may come into direct contact with food (tables, work surfaces, trays, meal trolleys, slicers, choppers, knives, etc.).

COMPOSITION

Disinfecting agents :

Ethyl alcohol (N°CAS 64-17-5: 72% m/m).

Also contains: Water, isopropanol, tert-butanol, sorbic acid. 96% of the total ingredients are of natural origin.

CHARACTERISTICS

Appearance: Clear, colourless liquid Odour: alcohol

pH (20°C): 4.0 - 7.0

INSTRUCTIONS FOR USE

PHAGO'SPRAY DASR 70 is ready to use.

Spray onto a single-use cloth and apply to the surface to be disinfected.

Do not rinse.

Spray directly onto previously cleaned surfaces from a distance of 20 to 30 cm at a rate of 15 to 20 ml/m².

Leave on for 2 minutes for a full spectrum of disinfection, or as required.

Frequency of application: refer to the defined hygiene plan.

TP1 - Biocidal products for human hygiene.

TP2 - Disinfectant for surfaces, materials, equipment and furniture not in direct contact with food or animal feed.

TP4 - Disinfectant for surfaces in contact with foodstuffs.

ANTIMICROBIAL ACTIVITY

		ACTION TIME
<u>Bactericide</u>	EN 1276	30 s
	EN 13727+A2	30 s
	EN 13697 of which <i>L. monocytogenes</i> et <i>S. enterica</i>	2 min
	EN 16615	1 min
	EN 17387	2 min
<u>Fungicide</u>	EN 1500 (3mL)	30 s
	EN 1650	1 min
	EN 13624	1 min
	EN 13697+A1	1 min
	EN 16615	2 min
<u>Mycobactericide</u>	EN 17387	2 min
	EN 14348	1 min
<u>Virucide</u>	EN 14476+A2 on Poliovirus	1 min
	EN 14476+A2 on Adenovirus, Norovirus and Vaccinia virus	30 s
	EN 16777 on Adenovirus and Norovirus	1 min

PRECAUTIONS FOR USE

Dangerous. For professional use only.

The safety data sheet is available on request.

Observe the precautions for use. Use biocides with care. Read the label and product information before use. Before any use, make sure that it is essential, particularly in areas frequented by the general public. Whenever possible, use alternative methods and products with the lowest risk to human and animal health and the environment.

Detergent certified by ECOCERT Greenlife according to the ECOCERT standard "Ecodetergent" available at <http://detergents.ecocert.com>

REFERENCES

SA0140980	6x750mL
SA0140990	2x5L
SA0141000	10L (not Ecocert certified)

PLEASE REFER TO THE SCIENTIFIC DOSSIER FOR MORE INFORMATION AND DETAILED REPORTS.



CHRISTEYNS

MEDICAL CARE

Christeyns France • 31 rue de la Maladrie • 44120 Vertou • Tél: +33 2 40 80 27 27
www.christeyns.com