

PHAGO



PHAGO'SPRAY DM

Hydroalcoholic disinfectant cleaner for disinfecting high surfaces



- Large spectrum of activity
- Foam or spray position
- · Leaves no trace when drying

USE

PHAGO'SPRAY DM is a professional disinfectant cleaner for disinfecting surfaces in high-risk environments. It can be sprayed on walls, ceilings and furniture in care and surgery departments. PHAGO'SPRAY DM can also be applied to single-use linen for rapid disinfection of small items of furniture, trolleys, external surfaces of non-invasive medical devices and small surfaces close to the patient.

COMPOSITION

Didecyldimethylammonium chloride (CAS n°7173-51-5; 0.14%) N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine (CAS n°2372-82-9; 0.25%) Ethanol (CAS n°64-17-5; 24.00% m/m)

TP2 - Disinfectant for surfaces, materials, equipment and furniture not in direct contact with food or feed.

TP4 - Surfaces in contact with food and animal feed.

CHARACTERISTICS

Other component: Perfume

Colour: colourless Fragrance citrus pH at 20°C: 9.5 – 11.0 Density at 20°C: 0.956 – 0.966

INSTRUCTION FOR USE

PHAGO'SPRAY DM is ready to use.

Spray onto a single-use cloth and/or apply to the surface to be disinfected. Do not rinse*. Leave on for 5 minutes for bactericidal activity or 15 minutes for full spectrum activity.

 In the case of disinfection of surfaces that may come into direct contact with foodstuffs, rinsing with drinking water is necessary (decree of 8 September 1999 and its amendments)

| | ANTIMICROBIAL ACTIVITY | | CONTACT TIME |
|--|---------------------------------------|---|----------------------------------|
| | <u>Bactericidal</u> | EN 13727 EN 13727+A2 SARM, A. baumanii, K. pneumoniae ESBL et OXA48 EN 13697 incl. S. enterica EN 16615 | 5 min 5 min 5 min 5 min |
| | <u>Yeasticidal</u> | EN 13624 <i>C. albicans</i> EN 13697 <i>C. albicans</i> EN 16615 <i>C. albicans</i> | 5 min 15 min 15 min |
| | <u>Mycobactericidal</u> | EN 14348 | 15 min |
| | Virucide against enveloped viruses | EN 14476+A2 Vaccinia virus EN 16777 Vaccinia virus EN 14476 Rotavirus | 15 min 15 min 5 min* |

^{*} In clean conditions

PRECAUTION FOR USE

Product for professional use only.

The safety data sheet is available on the publichem.com website. Wearing safety equipment is recommended.

Dangerous. Observe safety precautions. Use biocides with caution. Before use, read the label and product information. Before any use, make sure it is essential, particularly in areas frequented by the public. Wherever possible, give preference to alternative methods and products that present the least risk to human and animal health and to the environment.

Shelf life: 36 months

Shelf life after opening: until expiry

Close the container carefully after each use.

REFERENCES

60416 6x750mL 60414 2x5L

PLEASE REFER TO THE SCIENTIFIC DOSSIER FOR MORE DETAILED INFORMATION AND REPORTS

