



PHAGO

PHAGO'SURF 2D

Disinfectant detergent for floors, surfaces and equipment in critical environment



- o Large antimicrobial spectrum
- o Product combining detergency and disinfection
- o Optimization of toxicological data

USE

PHAGO'SURF 2 D is a concentrated liquid intended for cleaning and disinfecting floors, surfaces and materials in critical environment. It is an effective cleaner and a broad-spectrum disinfectant (bactericidal, mycobactericidal, fungicidal and active on viruses). PHAGO'SURF 2D can be used, with rinsing, on surfaces likely to come into direct contact with foodstuffs.

COMPOSITION

Didecylmethylammonium chloride (CAS No: 7173-51-5): 3.5% w/w
N- (3-aminopropyl) -N-dodecylpropane-1,3-diamine (CAS No: 2372-82-9): 5.5% w/w
Others components: nonionic surfactants <5%

CHARACTERISTICS

Color: colorless to yellow
pH at 20 ° C (pure product): 9.1 +/- 0.3
pH at 20 ° C (product diluted to 0.25% tap water): approximately 8.0
Density: 1.054 +/- 0.005

PT2. PT4.

INSTRUCTION FOR USE

Dilute a 20mL dose of PHAGO'SURF 2D in 8 liters of cold or lukewarm tap water, i.e. a 0.25% dilution.
Contact time: 15 minutes for bactericidal, yeastical, fungicidal and virucidal activity.
30 minutes for tuberculocidal activity.
Rinse with water any surfaces that may come into direct contact with food.

ANTIMICROBIAL ACTIVITY

CONTACT TIME

EFFICACY TESTS CARRIED OUT ON THE PRODUCT DILUTED AT 0.25%

<u>Bactericidal</u>	EN 13727 + A1 EN 13697 EN 16615	5 min* 15 min* 15 min*
<u>Yeastical</u>	EN 13697 <i>C.albicans</i> EN 13624 <i>C.albicans</i>	15 min* 15 min*
<u>Fungicidal</u>	EN 16615	15 min*
<u>Tuberculocidal</u>	EN 14348 <i>M.terrae</i>	30 min
<u>Active on viruses</u>	EN 14476 + A1 BVDV EN 16777 <i>Vaccina virus</i> Active on HBV, HCV, HIV, Herpes virus, Coronavirus, Orthomyxovirus, Infl uenza A/ H5N1 and Infl uenza A/H1N1	15 min* 15 min* 15min

*Dirty conditions

PRECAUTIONS FOR USE

Dangerous. Product reserved for professional use. Gloves are recommended.

The safety data sheet is available on the site www.quickfds.com.

Observe the precautions for use. Use biocides carefully. Before use, read the label and product information. Before any use, make sure that it is essential, especially in places frequented by the general public. Whenever possible, give preference to alternative methods and products presenting the lowest risk to human and animal health and to the environment.

REFERENCES

250x20mL doses
4x1L One2Dose
2x5L
20L

PLEASE REFER TO THE SCIENTIFIC DOSSIER FOR MORE DETAILED INFORMATION AND REPORTS.



CHRISTEYNS

MEDICAL CARE

Christeyns France • 31 rue de la Maladrie • 44120 Vertou • Tél: +33 2 40 80 27 27

www.christeyns.com