

# NOVA'CRM WD

# NOVASi

— Nova Science Corporation —

## Concentrated liquid detergent for medical and surgical instrumentation

### INDICATION

NOVA'CRM WD is a concentrated product which, when diluted in water, produces a liquid detergent for manual use, in ultrasound tanks and in machines for medical and surgical instrumentation, endoscopy equipment and medical devices.

### COMPOSITION

Non-ionic surfactants  
Sequestering agents  
Triethanolamine

### CHARACTERISTICS

Colour: amber  
pH at 20°C: 9.5 - 11.0  
Density at 20°C: 1.110 - 1.120

### INSTRUCTIONS FOR USE

Wear suitable personal protective equipment (see SDS).

In a manufacturing tank, make a 50% dilution of NOVA'CRM WD in water, i.e. 1000kg of NOVA'CRM WD (1 IBC) + 1000kg of osmosis water.

Shake for 10 minutes for complete homogenisation.  
Package the resulting product in 5L and 10L cans.

### PRECAUTIONS FOR USE

Wear eye protection, face protection, protective clothing and protective gloves.  
The safety data sheet is available on request.

### STORAGE

NOVA'CRM WD can be stored for 36 months in its original packaging at room temperature, protected from light.

### PACKAGING

IBC 1000L

PLEASE REFER TO THE SCIENTIFIC DOSSIER FOR MORE INFORMATION AND DETAILED REPORTS.

Rev – 06/2024

#### NOVASi BV

Oude Mechelsestraat 165, 1853 Grimbergen, Belgium  
VAT: BE079766663  
info@novasi.be | www.novasi.be